

OCL® Plaster of Paris Bandages
without core, Green Label (formerly M-PACT®)

Casting Impression

Vendor:

BSN Medical Inc.

Features & Benefits:

- Made from woven leno cotton gauze impregnated with plaster of Paris
- Fast setting plaster of Paris used ensures fractures can be rapidly immobilised
- No central core
- Latex free

Specifications:

- Setting time: Use clean water (22–25°C). Cold water retards setting time. Hot water will hasten initial set
- Use: Immerse bandage in water at 45° slant for 3–5 seconds only
- Approximate initial setting time 2–4 minutes
- Approximate final set time 48 hours

Indications:

- External circumferential immobilisation of fractures
- Emergency trauma splinting
- Total contact casting, diabetic feet

Contraindications:

- Caution: exotherm reaction of the plaster during the setting phase can generate heat—avoid direct contact with the skin or damage may occur
- Do not use where there is a risk of swelling

Call Points:

- Emergency department
- Fracture clinic
- Orthotic technician
- Prosthetic technician

Warranty:

- Against manufacturer defects



13H7

OCL® Plaster of Paris Bandages » BSN Medical Inc.

Item No.	Description	Size (Metric)	Size (Imperial)	Unit
BSN-41923X-BX	Plaster Bandage without Core, Green Label (2–4 Minute Set)	5 cm X 2.7 m	2" X 3 yds	12 rolls/bx
BSN-41923X-CS	Plaster Bandage without Core, Green Label (2–4 Minute Set)	5 cm X 2.7 m	2" X 3 yds	72 rolls/cs 6 bx/cs
BSN-41933X-BX	Plaster Bandage without Core, Green Label (2–4 Minute Set)	7.5 cm X 2.7 m	3" X 3 yds	12 rolls/bx
BSN-41933X-CS	Plaster Bandage without Core, Green Label (2–4 Minute Set)	7.5 cm X 2.7 m	3" X 3 yds	72 rolls/cs 6 bx/cs
BSN-41935X-BX	Plaster Bandage without Core, Green Label (2–4 Minute Set)	7.5 cm X 4.6 m	3" X 5 yds	12 rolls/bx
BSN-41935X-CS	Plaster Bandage without Core, Green Label (2–4 Minute Set)	7.5 cm X 4.6 m	3" X 5 yds	72 rolls/cs 6 bx/cs
BSN-41945X-BX	Plaster Bandage without Core, Green Label (2–4 Minute Set)	10 cm X 4.6 m	4" X 5 yds	12 rolls/bx
BSN-41945X-CS	Plaster Bandage without Core, Green Label (2–4 Minute Set)	10 cm X 4.6 m	4" X 5 yds	72 rolls/cs 6 bx/cs
BSN-41955X-BX	Plaster Bandage without Core, Green Label (2–4 Minute Set)	12.5 cm X 4.6 m	5" X 5 yds	12 rolls/bx
BSN-41955X-CS	Plaster Bandage without Core, Green Label (2–4 Minute Set)	12.5 cm X 4.6 m	5" X 5 yds	48 rolls/cs 4 bx/cs
BSN-41963X-BX	Plaster Bandage without Core, Green Label (2–4 Minute Set)	15 cm X 2.7 m	6" X 3 yds	12 rolls/bx
BSN-41963X-CS	Plaster Bandage without Core, Green Label (2–4 Minute Set)	15 cm X 2.7 m	6" X 3 yds	48 rolls/cs 4 bx/cs
BSN-41965X-BX	Plaster Bandage without Core, Green Label (2–4 Minute Set)	15 cm X 4.6 m	6" X 5 yds	12 rolls/bx
BSN-41965X-CS	Plaster Bandage without Core, Green Label (2–4 Minute Set)	15 cm X 4.6 m	6" X 5 yds	48 rolls/cs 4 bx/cs
BSN-41985X-BX	Plaster Bandage without Core, Green Label (2–4 Minute Set)	20 cm X 4.6 m	8" X 5 yds	12 rolls/bx
BSN-41985X-CS	Plaster Bandage without Core, Green Label (2–4 Minute Set)	20 cm X 4.6 m	8" X 5 yds	36 rolls/cs 3 bx/cs