

# PRODUCTS



*Serving Prosthetists and Orthotists*

## Centri® Dream Skin Instructions



### Step 1

It is necessary to use a foam cover underneath the cosmetic skin. Powder liberally to make the skin slide more smoothly over the foot and ankle.



### Step 2

Sprinkle a marginal amount of powder in skin. Close proximal end and shake to coat interior. Pour out excess.




### Step 3

Be sure toes and heel are oriented correctly. Gather and pull Dream Skin over the foot. Avoid air pockets and wrinkles on the plantar surface.



### Step 4

Skin should be stretched just enough to remove slack and wrinkles. With skin in correct position, pull over proximal edge. CAUTION: Overstretching will cause tension on the skin and a greater risk of splitting.

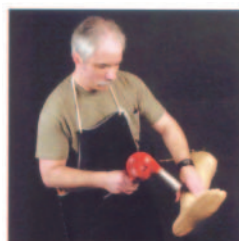

### Step 5

Heat lightly to conform skin to contours and prevents wrinkles. Allow skin to float over prosthesis. CAUTION: Be careful not to over heat the skin. Over heating will damage the vinyl, deform underlying foam, and void the warranty.



### Step 6

It is unnecessary to glue skin in any area other than the proximal brim. Prepare skin to be glued by cutting away excess. Cut line should be 2 inches above proximal brim and should follow curve.

*Serving Prosthetists and Orthotists*



**Step 7**

Apply a thin line of Super Glue to the top edge of the brim in small sections (one to two inches at a time).



**Step 8**

Fold skin over edge and apply pressure until dry. Alternate medial and lateral sides to avoid wrinkles. Allow glue to dry completely.



**Step 9**

Using a sharp scalpel, carefully trim unattached skin along the inside edge of the socket.



**Step 10**

If adjustments to the foot are necessary in the future, simply grind along the top of the socket and reflect the skin. The same Dream Skin may be re-used following this method of fabrication.